Rely+On™ Virkon™

High Level Surface Disinfectant

LANXESS

Virucidal / Bactericidal / Fungicidal

FIRST AID MEASURERS

General Advice: Never give anything by mouth to an unconscious person. When symptoms persist or in all cases of doubt seek medical advice.

Inhalation: Remove from exposure, lie down. If victim has stopped breathing: Artificial respiration and/or oxygen may be necessary. Consult a physician.

Skin Contact: Take off all contaminated clothing immediately. Wash off immediately with plenty of water. Wash contaminated clothing before re-use. Consult a physician.

Eye Contact: Rinse immediately with plenty of water and seek medical advice

Ingestion: Do NOT induce vomiting. If conscious, give small quantities of water to drink. Call a physician immediately.

PRECAUTIONS

As with all disinfectants, wash splashes of solution from skin and eyes immediately. Do not mix with other chemical solutions, except those recommended by the manufacturer. Do NOT prepare concentrated solutions for later dilution.

Rely+On™ Virkon™ solutions must not be used on medical devices.

Additional precautions are provided on the container label; always read prior to use. Not for use on food contact surfaces. Not to be used on Medical Devices. For professional use only. Keep away from children. Not to be taken by mouth.

STORAGE

Store closed containers in a cool, dry place. Store prepared solution out of direct sunlight at a ambient temperature (15-25°C).

DISPOSAL

Unused solution of Rely+On™ Virkon™ may be disposed of via normal liquid waste systems, such as sinks and drains connected to the municipal sewer system in accordance with local authority regulations. Do NOT dispose of to storm water drains or any other unprotected waste systems. Dispose of waste packaging and/or waste powder in accordance with local requirements.

APPLICATIONS AND INSTRUCTIONS FOR USE

APPLICATIONS

Rely+On™ Virkon™ is suitable for disinfection of surfaces that transmit infectious diseases in many environments. These include healthcare institutions, pathology and clinical research laboratories, food production, preparation and catering establishments, hotel and residential settings and transport.

Application of Rely+On™ Virkon™ solution is especially effective for routine preventative disinfection and for terminal decontamination in an infectious disease outbreak. The frequency of routine use will depend on the infection control practices of the facility in question. Consult your local representative for further advice.

Rely+On™ Virkon™ is highly tolerant of organic challenge with its biocidal efficacy retained in 'dirty' conditions. Independent testing of Rely+On™ Virkon™ has proven efficacy against important infectious disease pathogens. Data supporting efficacy for Rely+On™ Virkon™ viral, bacterial and fungal pathogens of interest to infection control, including HIV, hepatitis B and C, MRSA, VRE and Salmonella.

Examples of specific organisms relevant to human infectious diseases are listed below.

Viruses

Family	Example(s)	Family	Example(s)
Herpesviridae	Herpes Simplex	Caliciviridae	Norwalk-like virus
Adenoviridae	Human Adenovirus	Coronaviridae	SARS virus
Picornaviridae	Polio Type 1	Paramyxoviridae	AI H5N1
Hepadnaviridae	Hepatitis B	Retroviridae	HIV Type 1
Flaviviridae	Hepatitis C	Orthomyxoviridae	Swine Influenza A (H1N1)

Bacteria

E. Coli O157	Legionella	Listeria	Salmonella
Campylobacter	Klebsiella pneumoniae	Streptococci	Pseudomonas
Shigella	Staphylococci (incl. MRSA)	Clostridium sporogenes	Enterococcus faecalis (VRE)

Fungi

Candida albicans	Trichophyton sp.	Saccharomyces cerevisiae	Aspergillus fumigatus
------------------	------------------	--------------------------	-----------------------

SURFACES

Hard Surfaces: Floors, laboratory benches, work tops, bed frames, tables, sinks, baths, toilets, etc. in healthcare institutions, laboratories, food and catering establishments and public environments.

1. Prepare a 1% solution as per directions on label or shown opposite.
2. Small areas: apply Rely+On™ Virkon™ solution via a spray bottle.
Large areas: apply Rely+On™ Virkon™ solution with a cloth or mop for floors.
3. Leave solution on the surface for 10 minutes, wipe dry with absorbent paper towel or cloth to remove any residue and then discard to waste.

LABORATORY EQUIPMENT (Centrifuges, Clinical Analysers, pipettes etc.)

1. Prepare 1% solution as directed.
2. a) Equipment: wipe over surfaces which may be contaminated with infected material. Sample probes and tubing can be decontaminated by aspiration Rely+On™ Virkon™ of solution. Always check compatibility of Rely+On™ Virkon™ with components with manufacturer of equipment before proceeding.
b) Discard jars: fill jar or container with fresh solution daily.

BODY FLUID SPILLAGES (blood, urine, vomitus, etc)

Hard surfaces: Dispense Rely+On™ Virkon™ powder directly on the spillage until all liquid is absorbed. Scrape or sweep damp powder into container for disposal. After removal of damp powder, apply a 1% solution to the affected area and items used for disposal. Allow 10 minutes contact time and wipe to remove solution/residue with cloth or paper towels.

PREPARATION OF SOLUTION

Add 5 g of powder per 500 ml of lukewarm water. Mix thoroughly until completely dissolved to form a pink coloured solution.

Powder	Water
5 g	500 ml
10 g	1 Litre
20 g	2 Litres
50 g	5 Litres

1% Rely+On™ Virkon™ solutions are stable for up to 5 days. Recommended good practice is that solutions are replenished and discarded daily.

Manufactured by

Antec International Limited
Windham Road, Chilton Industrial Estate,
Sudbury, Suffolk, CO10 2XD, United Kingdom
Tel: +44 (0) 1787 377305 Fax: +44 (0) 1787 310846,
relyondisinfection@lanxess.com
www.relyondisinfection.com

Emergency Telephone Number: +49 214 30 99300
(Size CHEMPARK Leverkusen)

Made in England

©2017 LANXESS. Rely+On™Virkon™, LANXESS™ and the LANXESS Logo are trademarks of LANXESS Deutschland GmbH or its affiliates. All trademarks are registered in many countries worldwide.

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